

Adhesion Prophylaxis for Plate Osteosynthesis

Example of Use

Male patient, age 78

Adhesion and infection prophylaxis for insufficient soft tissue coverage of the plate

Dr. Claudia Schuster

Department of Trauma, Hand, and Restorative Surgery
Friedrich Schiller University Hospital, Jena



REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

 **RESORBA**[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

 **LGA InterCert**
Zertifiziertes QS-System
Medizinproduktehersteller
93442 EWG Ascham II

 **LGA InterCert**
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Open PIP Dislocation following a Chainsaw Injury

Example of Use

Male patient, age 62

Separation layer benefiting both joint restoration and infection prophylaxis following traumatic cartilage damage

Dr. Claudia Schuster

Department of Trauma, Hand, and Restorative Surgery
Friedrich Schiller University
Hospital, Jena



REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

 **RESORBA**[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

 **LGA InterCert**
Zertifiziertes QS-System
Medizinproduktehersteller
93042 EWG Anhang II

 **LGA InterCert**
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Adhesion Prophylaxis for Flexor Tendons Rupture

Example of Use

Male patient, age 42

Initial treatment of a 3-week-old flexor tendon rupture

Dr. Claudia Schuster

Department of Trauma, Hand, and Restorative Surgery
Friedrich Schiller University Hospital, Jena



REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

 **RESORBA**[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

 **InterCert**
Zertifiziertes QS-System
Medizinproduktlinie
93/42 EWG Anhang II

 **InterCert**
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Adhesion and Infection Prophylaxis for Gunshot Wound


Example of Use

Male patient, age 36

Revision operations
Gunshot wound, 1st session
2-stage flexor tendon replacement
with silicone spacer after rerupture

Dr. Claudia Schuster

Department of Trauma, Hand, and
Restorative Surgery
Friedrich Schiller University
Hospital, Jena

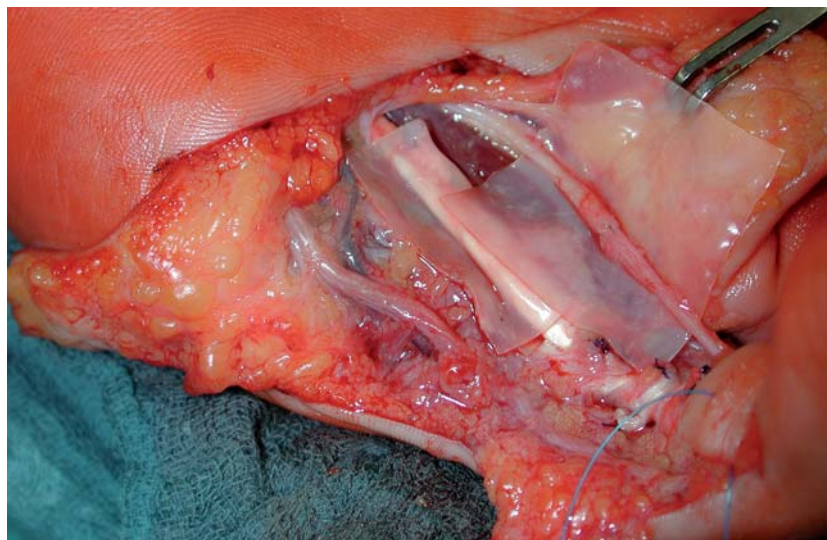

REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

RESORBA[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

LGQA / InterCert
Zertifiziertes QS-System
Medizinprodukteherstellung
93042 EWG Aachang II

LGQA / InterCert
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Adhesion and Infection Prophylaxis for Prostheses

Example of Use

Female patient, age 68

SBI prosthesis for Heberden's nodes

Dr. Claudia Schuster

Department of Trauma, Hand, and Restorative Surgery
Friedrich Schiller University Hospital, Jena



REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

RESORBA[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

LGQA / InterCert
Zertifiziertes QS-System
Medizinproduktlinie
93/42 EWG Anhang II

LGQA / InterCert
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Adhesion Prophylaxis for Neurolysis



Example of Use

Male patient, age 49

Neurolysis of the radial nerve
following humeral shaft fracture

Dr. Claudia Schuster

Department of Trauma, Hand, and
Restorative Surgery
Friedrich Schiller University
Hospital, Jena

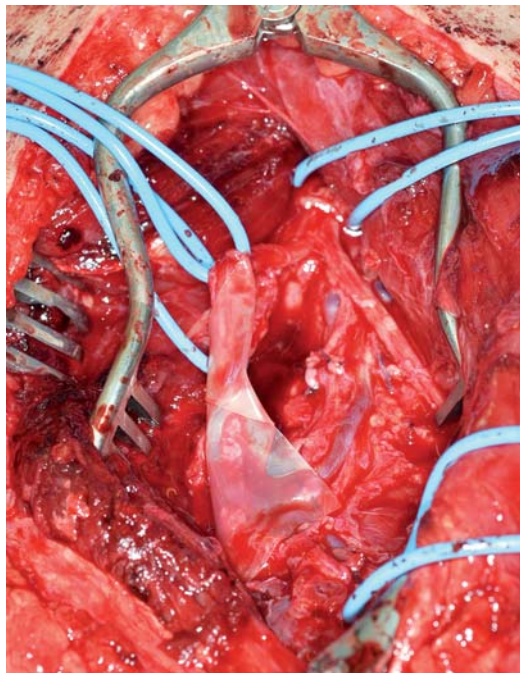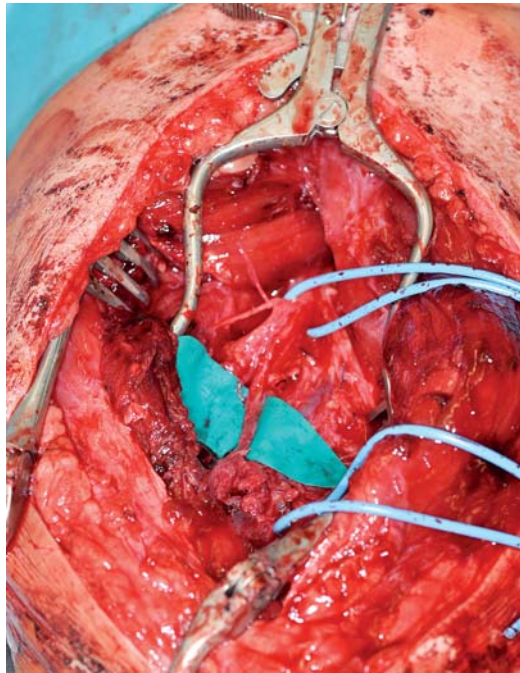

REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

 **RESORBA**[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

 **LGA/InterCert**
Zertifiziertes QS-System
Medizinproduktehersteller
9342 EWG Anhang II

 **LGA/InterCert**
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Infection Therapy and Prophylaxis for Synovitis



Example of Use

Female patient, age 64

Synovitis following annular ligament
desmotomy
Plus CTS

Dr. Claudia Schuster

Department of Trauma, Hand, and
Restorative Surgery
Friedrich Schiller University
Hospital, Jena

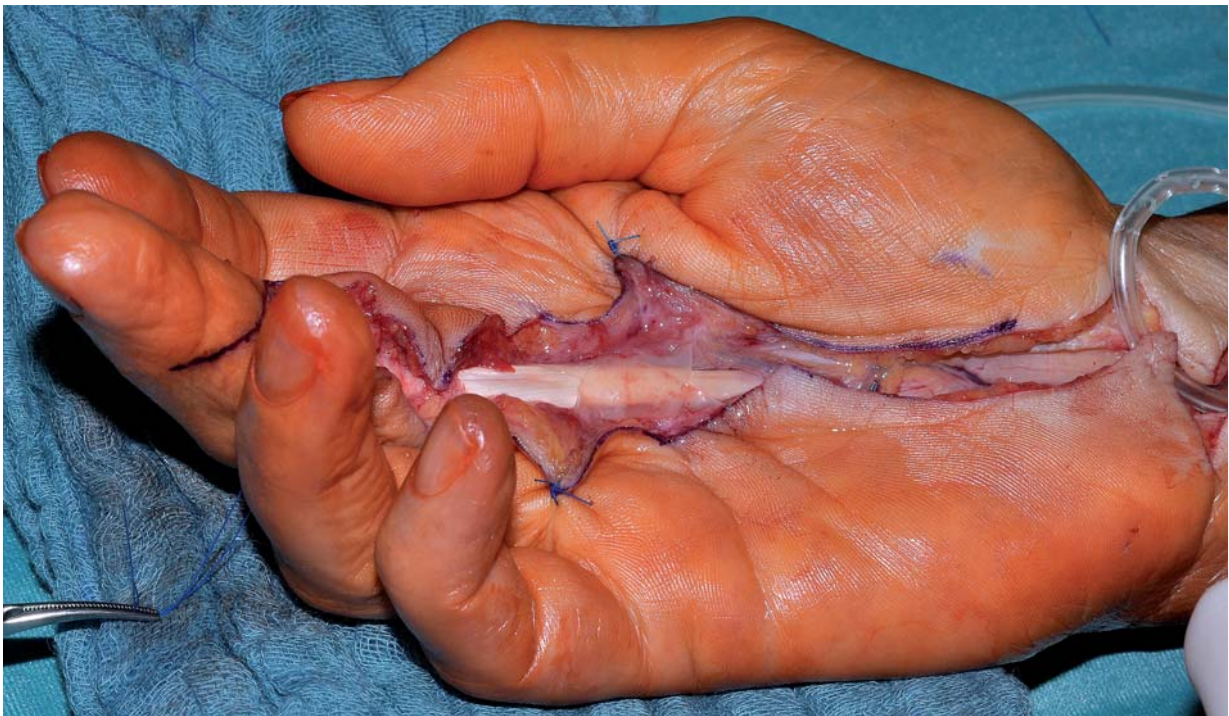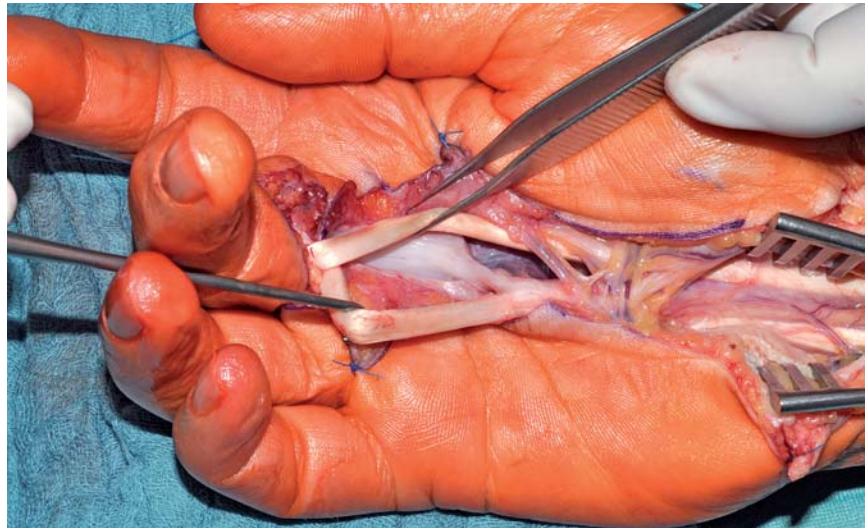

REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

RESORBA[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

LGQA / InterCert
Zertifiziertes QS-System
Medizinproduktüberwachung
9342 EWG Anhang II

LGQA / InterCert
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007

Revision Operations

Example of Use

Female patient, age 76

Diagnosis:
Extensor tendon adhesions following
plate osteosynthesis



Removal of plate and tenolysis of the
extensor tendon (image 1)

Suture of the extensor tendon with
gliding layer absent (image 2)

Wound closure (image 3)

Dr. Claudia Schuster

Department of Trauma, Hand, and
Restorative Surgery
Friedrich Schiller University
Hospital, Jena

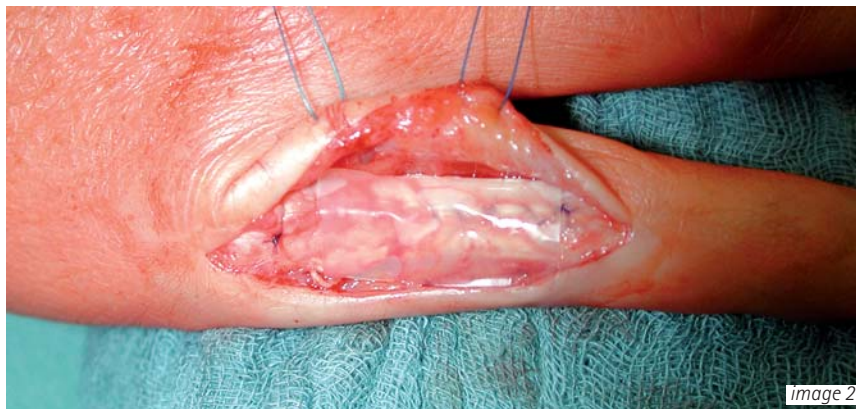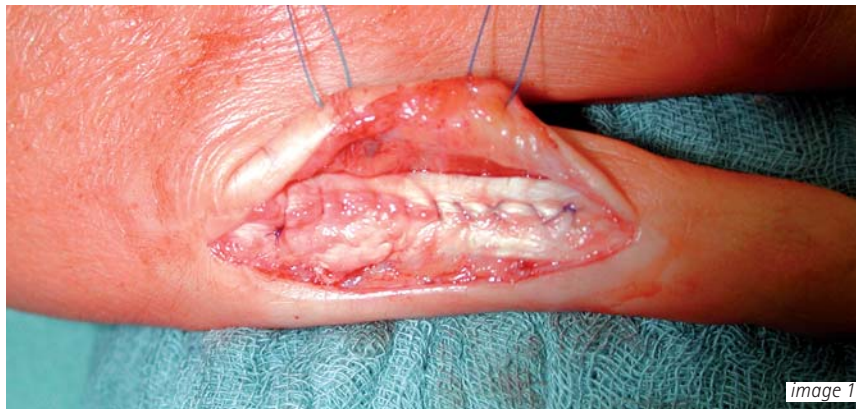

REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

RESORBA[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com



Infection Prophylaxis for Animal Bites


Example of Use

Female patient, age 54

Finger ischemia, fracture of the proximal phalanx and exposed tendons following a dog bite

Dr. Claudia Schuster

Department of Trauma, Hand, and Restorative Surgery
Friedrich Schiller University Hospital, Jena

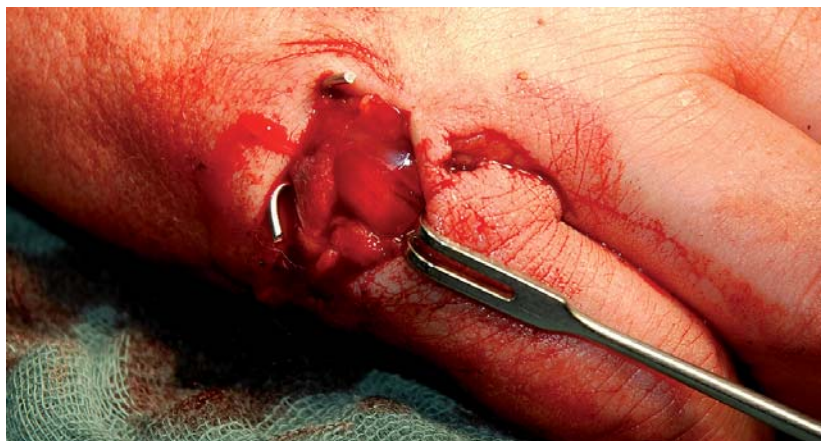

REF	Size	Contents of Package
GF 25	2.5 x 2.5 cm	1 film / package
GF 255	2.5 x 5 cm	1 film / package

1 cm² of film contains:
5.6 mg collagen of equine origin
4 mg gentamicin sulfate equivalent to
2.20-2.86 mg gentamicin base

CE 1275

 **RESORBA**[®]
REPAIR AND REGENERATE

RESORBA Wundversorgung GmbH & Co. KG, Am Flachmoor 16, 90475 Nuremberg, Germany
Tel. +49 9128 / 91 15 0, Fax +49 9128 / 91 15 91, infomail@resorba.com, www.resorba.com

 **InterCert**
Zertifiziertes QS-System
Medizinproduktehersteller
93/42 EWG Anhang II

 **InterCert**
Zertifiziertes QM-System
EN ISO 13485:2003 + AC:2007